


CHG 238 Multiple Battery Charger

12 Unit Charger. For use with BAT AP97A NiMH Rechargeable Batteries.

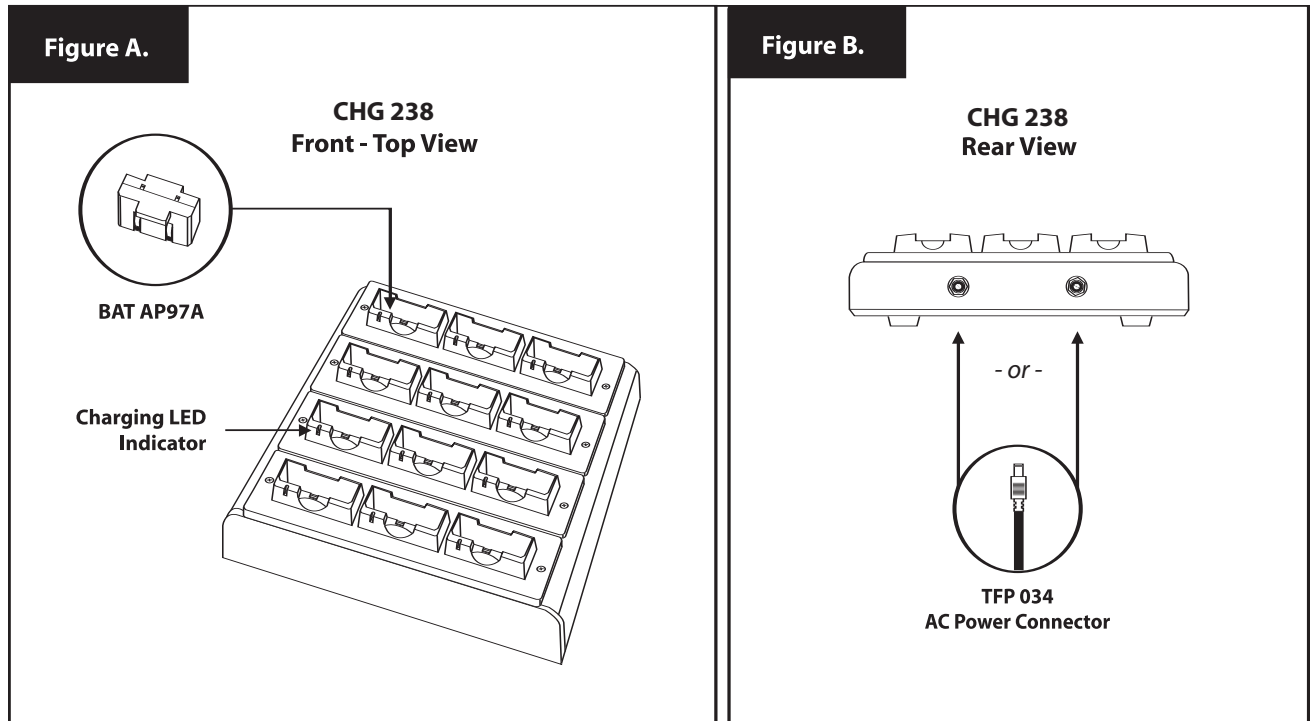

Instructions for Use and Care

- 1.) Plug the TFP 024 power supply into the AC wall outlet. Plug the power connector into either jack on the rear of the CHG 238 unit (Figure B.).

WARNING: DO NOT ATTEMPT TO CHARGE NON-RECHARGEABLE BATTERIES.
Non-rechargeable batteries, such as alkaline and carbon zinc, may heat up and burst, causing possible injury and damage to the equipment.

- 2.) Insert the AP97A NiMH rechargeable battery into the charging dock as illustrated in Figure A above. A red LED indicator will illuminate when charging has started (Figure A).
 - You may charge up to 12 battery units per CHG 238 charger.
 - It will take approximately 14 hours to fully charge the batteries.

CHG 238 Multiple Battery Charger Specifications

Size: 6" x 5 3/4" x 1 3/4" (150 mm x 145 mm x 45 mm)

Weight: 535g (1.18 lbs) with power supply

Color: Black

Power Input: 115 AC input

Power Supply: AC adaptor, 12VDC - 200 mA

Charging Rate: Overnight Charge: 14 mA
Trickle/Maintain: 8 mA

Charging Time: 14 hours nominal

Unit Capacity: 12 Units

Indicators: Individual Charging Indicators: Red LEDs

RECYCLING INSTRUCTIONS



Help Williams Sound protect the environment! Please take the time to dispose of your equipment properly.



Product Recycling for Customers in the European Union:

Please do NOT dispose of your Williams Sound equipment in the household trash. Please take the equipment to a electronics recycling center; OR return the product to the factory for proper disposal.



LIMITED WARRANTY

Williams Sound products are engineered, designed and manufactured under carefully controlled conditions to provide you with many years of reliable service. Williams Sound warrants the CHG 238 against defects in materials and workmanship for ONE (1) year. During the first year from the purchase date, we will promptly repair or replace the CHG 238.

Microphones, earphones, headphones, rechargeable batteries, chargers, cables, power supplies, carry cases, and all other accessory products carry a 90-day warranty.

WILLIAMS SOUND HAS NO CONTROL OVER THE CONDITIONS UNDER WHICH THIS PRODUCT IS USED. WILLIAMS SOUND, THEREFORE, DISCLAIMS ALL WARRANTIES NOT SET FORTH ABOVE, BOTH EXPRESS AND IMPLIED, WITH RESPECT TO THE CHG 238, INCLUDING BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. WILLIAMS SOUND SHALL NOT BE LIABLE TO ANY PERSON OR ENTITY FOR ANY MEDICAL EXPENSES OR ANY DIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES CAUSED BY ANY USE, DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT, WHETHER A CLAIM FOR SUCH DAMAGES IS BASED UPON WARRANTY, CONTRACT, TORT OR OTHERWISE. THE SOLE REMEDY FOR ANY DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT IS REPLACEMENT OF THE PRODUCT. NO PERSON HAS ANY AUTHORITY TO BIND WILLIAMS SOUND TO ANY REPRESENTATION OR WARRANTY WITH RESPECT TO THE CHG 238. UNAUTHORIZED REPAIRS OR MODIFICATIONS WILL VOID THE WARRANTY.

The exclusions and limitations set out above are not intended to, and should not be construed so as to contravene mandatory provisions of applicable law. If any part or term of this Disclaimer of Warranty is held to be illegal, unenforceable or in conflict with applicable law by a court of competent jurisdiction, the validity of the remaining portions of this Disclaimer of Warranty shall not be affected, and all rights and obligations shall be construed and enforced as if this Limited Warranty did not contain the particular part or term held to be invalid.

If you experience difficulty with your system, call Toll-Free for Customer Assistance:

1-800-843-3544

If it is necessary to return the system for service, your Customer Service Representative will give you a Return Authorization Number (RA) and shipping instructions.

Pack the system carefully and send it to:

Williams Sound Corp.
Attn: Repair Dept.
10321 West 70th Street
Eden Prairie, MN 55344



WILLIAMS SOUND®

Helping People Hear

10321 West 70th St., Eden Prairie, MN 55344 U.S.A.
800.843.3544 | 952.943.2252 | FAX: 952.943.2174
www.williamssound.com